

1 Open IntelliSwab Connect

Create an Account [HERE](#)


Quick Start Guide: IntelliSwab® Connect App

How to access the app, set up an account, record, and review test results on a smartphone, tablet or computer




2 Request Activation Code

Enter full name & email address, then click **SAVE** to request an activation code.

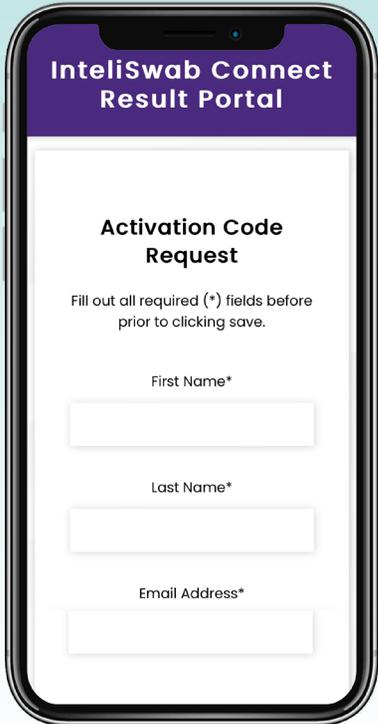

3 Verify Account


Check email address and activate account by clicking on the link.

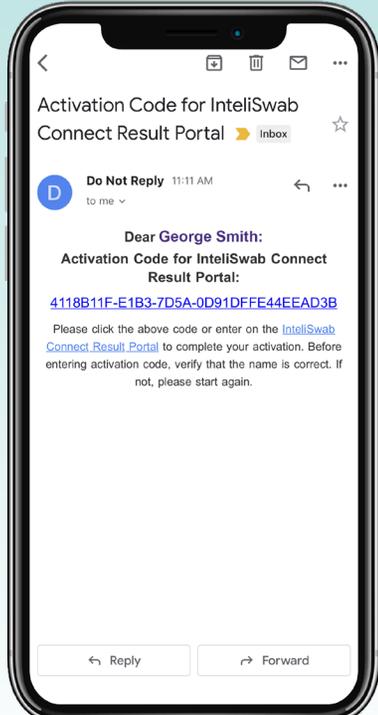

Can't access link in the email? Refer to the [FAQ](#)


4 Create User ID & Password

Create a User ID and Password that follow guidelines, then click **SAVE**.

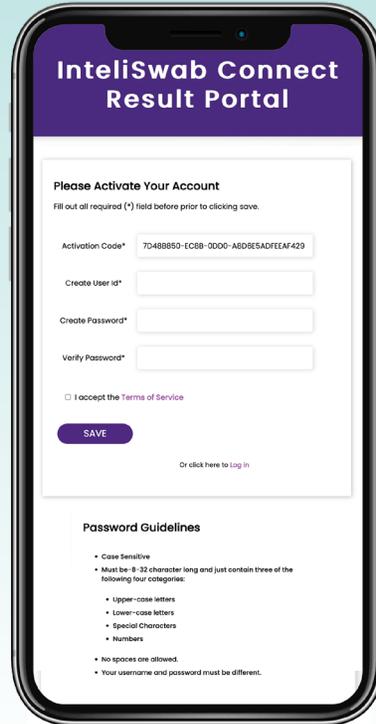

5 Log In


Enter User ID and Password. Then click **Log In**. *Note: Log in [HERE](#)*

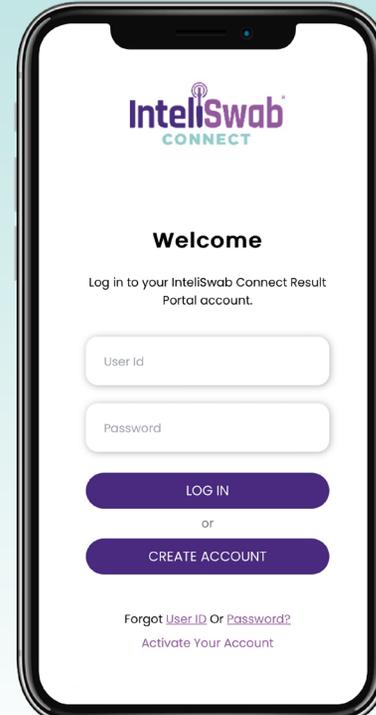

Bookmark this page to easily access IntelliSwab Connect in the future


6 Complete Registration

Fill out personal information (DOB, Address, Phone Number, Race, Ethnicity, and Sex). Then click **SAVE**.

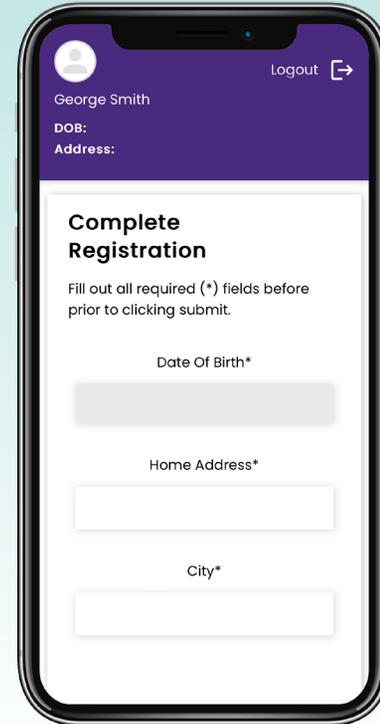

7 Enter IntelliSwab Test Result

Click on **Enter Result**, record IntelliSwab test result, then click **SAVE**.

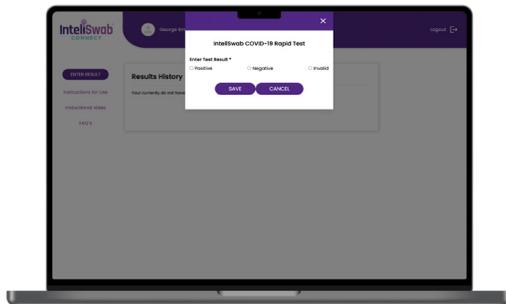

Results History Dashboard

The dashboard will reveal the user's test result history. Click on **CLICK HERE TO VIEW RESULTS** to obtain a PDF of the recorded test result.

Note: To view result pop-up blockers may need to be disabled, refer to the [FAQ](#)

