
POSITIVE TEST RESULTS ADDITIONAL REFERENCE IMAGES

InteliSwab[™]
COVID-19 RAPID TEST • Rx

This card provides additional examples of what a positive result could look like. Use the **Instructions for Use** included in the box to complete your test. Upon completion, if you think you have a positive result but are unsure, please compare your test to the images below.

EXAMPLES OF POSITIVE TEST RESULTS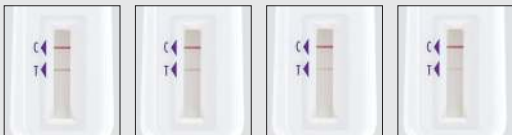

These photos show how faint the line next to the “T” may be. These are all positive test results. In some cases, the line next to the “C” may be very faint.

Look very closely!

The bottom line may be **very faint**.

Any line next to the “T” means you may have COVID-19, even if the line is very faint.

THE TEST RESULT IS ALSO POSITIVE IF:

There is a line next to the “T” and **NO** line next to the “C”

If your test does not look like any of these, see **Instructions for Use** for more examples of test results. **Be sure to follow all instructions.**

This product has not been FDA cleared or approved; but has been authorized by FDA under an EUA; This product has been authorized only for the detection of proteins from SARS- CoV-2, not for any other viruses or pathogens; and, This product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

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