


Quick Start Guide: IntelliSwab[®] Connect App

How to download the app, set up your account, & submit your test results to public health

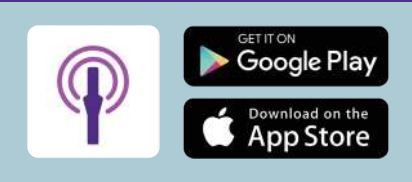


1 Download the app from **Google Play** or the **iOS App Store**.




2 During the installation process, we suggest accepting all notifications. The settings can be changed at a later date, e.g. to allow location only while using the app.


3 Select the **Sign Up Now** button and enter your information. Select **Create Account** and an email will be sent for verification; click the link to verify.

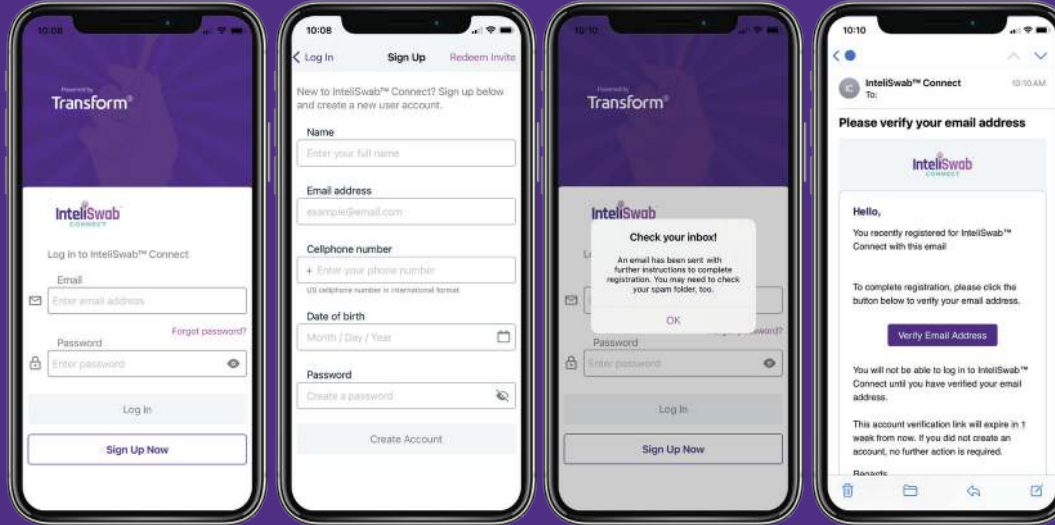

4 Log back into the app and confirm consent.

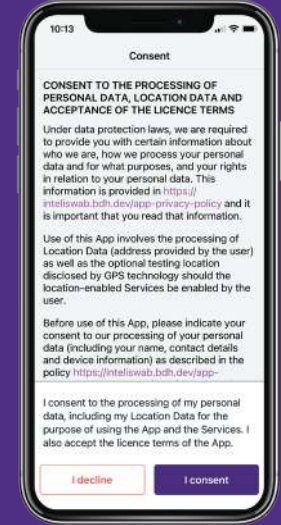

5 Select **Record A Test**, select the **Test Result**, take a photo of the test device, and enter all your information in the fields of the subject tab. The **Submit Test** button will turn purple. Click the **Submit Test** button.

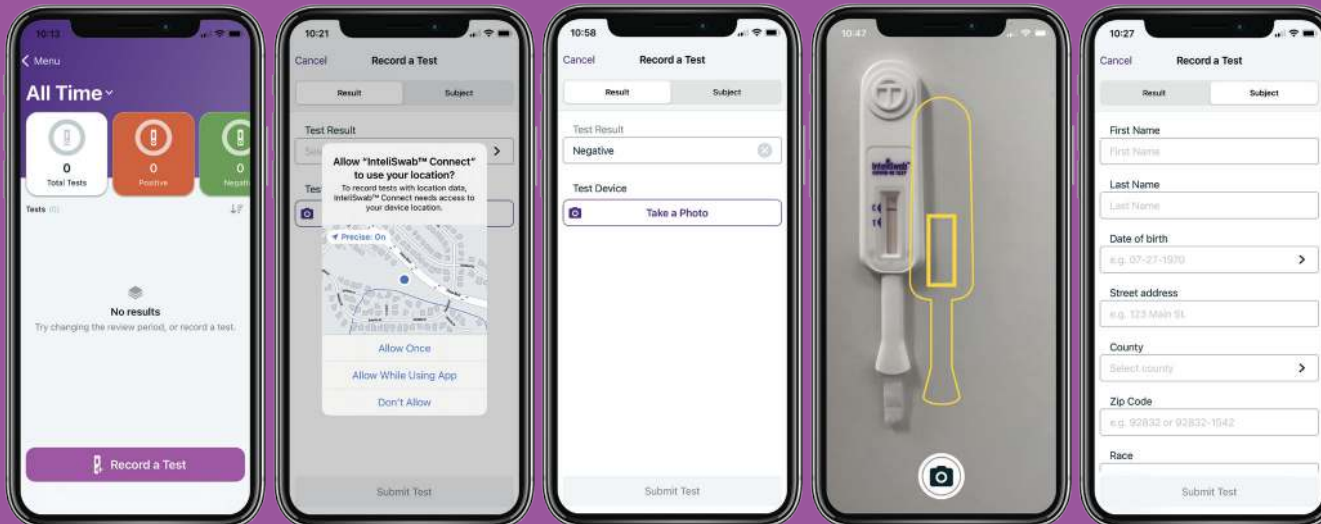

6 The dashboard will reveal your test result history. Click **Menu** to learn more & logout.

